

# EVACUATION SLYDE

## BARIATRIC EVACUATION SLYDE



Evacuation Equipment

## FAQs

- Q.** How much do the Belluscura Evacuation Slyde products weigh?
- A.** The Evacuation Slyde weighs just 9 pounds. The Bariatric Evacuation Slyde weighs 11 pounds.
- 
- Q.** What material is used in the Evacuation Slyde and Bariatric Evacuation Slyde?
- A.** They are constructed of durable, yet light, corrugated polypropylene, up to 12 millimeters thick.
- 
- Q.** How do I store Belluscura Evacuation Slyde products so I can access them quickly?
- A.** The Belluscura Evacuation Slyde folds flat and can be stacked. Up to 15 can be stored in the wall-mounted Belluscura Evacuation Slyde Sleeve. Up to 15 of the wider, Bariatric Evacuation Slyde version can be stored in the Bariatric Evacuation Slyde Sleeve. The Evacuation Slyde products must be stored at 50° – 90° F.
- 
- Q.** How many people are required to control the stair descent of an Evacuation Slyde?
- A.** Based on the weight of the patient, a minimum of 2 to 7 people can control the stair descent of patients. For more information, please refer to the operations manual.
- 
- Q.** Is the Belluscura Evacuation Slyde reusable?
- A.** The intended use for the Evacuation Slyde is for a single evacuation incident. However, the Evacuation Slyde can be reused during facility training exercises if prior to each use, the Evacuation Slyde is checked for damage and functionality. Should the device ever come in contact with fluids, it should not be reused.
- 
- Q.** Does Belluscura offer training for the Evacuation Slyde?
- A.** Belluscura has several options for training. To learn more about available training, please contact your Evacuation Consultant. Belluscura sales representatives can be located at [customerservice@belluscura.com](mailto:customerservice@belluscura.com)
- 
- Q.** What is the rating of the Belluscura Stairwell Belay System?
- A.** The equipment is rated at 22.2 kilonewtons or 5000 pounds. The system uses 30 feet (9 meters) of rope, two carabiners and a belay friction device to assist with control descent.
- 
- Q.** What is the purpose of the Belluscura Stairwell Belay Box?
- A.** Installed in the stairwell of every other floor, the Stairwell Belay Box stores the Stairwell Belay. When the included anchor is installed, it provides a secure anchor point for the Stairwell Belay System.



## CONTACT INFORMATION

email: [customerservice@belluscura.com](mailto:customerservice@belluscura.com)  
tel: +1 786-703-7996  
toll free: +1 844-883-0940

Belluscura, LLC.  
15455 Dallas Pkwy., Suite 600  
Addison, TX 75001  
United States  
tel: +1 786-703-7996  
toll free: +1 844-883-0940

Belluscura, Ltd.  
Oxford Center for Innovation  
New Road, Oxford OX1 1BY  
United Kingdom  
tel: +44 (0) 1865 261506  
toll free: +1 844-883-0940