Evacuation Slyde
REF Model 6400-000-000
Bariatric Evacuation Slyde
REF Model 6410-000-000

ENGLISH

Operations Manual

Toll Free Customer Service
International: +1 844-883-0940

For parts or technical assistance:
USA: +1 786-703-7996

Table of Contents

Symbols and Definitions	2
Introduction.....	2
Intended Use	2
Specifications	3
Product Inspection.....	4
Assembly Instructions.....	4
Transporting An Occupant Using Slyde.....	5
Descending Stairs Using Evacuation Slyde	6
Descending Stairs Using Bariatric Evacuation Slyde	6
Additional Parts and Accessories List.....	6
Operator Belay Configuratio.....	7
Warranty	8

Symbols and Definitions

The words **WARNING**, **CAUTION** and **NOTE** carry special meanings and should be carefully reviewed.

Safe Working Load

Single Use

WARNING

Alerts the reader about a situation, which if not avoided, could result in death or serious injury. It may also describe potential serious adverse reactions and safety hazards.

CAUTION

Alerts the reader of a potentially hazardous situation, which if not avoided, may result in minor or moderate injury to the user or occupant or damage to the equipment or other property. This includes special care necessary for the safe and effective use of the device and the care necessary to avoid damage to a device that may occur as a result of use or misuse.

NOTE

This provides special information to make maintenance easier or important instructions clearer.

Introduction

This manual will assist you with the operation and maintenance of Evacuation Slyde/Bariatric Evacuation Slyde [Slyde]. Thoroughly read this manual before using the equipment or beginning maintenance. Your staff should be trained on safe operation with written procedures.

INTENDED USE

Slyde is an emergency sled for safely evacuating non-ambulatory residents or patients [occupant] weighing up to 500 lb (227 kg) using the Evacuation Slyde or 800 lb (363 kg) using the Bariatric Evacuation Slyde from any multi-floor building using the stairway.

WARNING

- Do not modify the Slyde. Modifying the equipment can cause unpredictable operation resulting in injury to the occupant or operator. Modifying the Slyde will also void its warranty.
- Slyde is not designed for use with suspected or known cervical, spinal, or fracture injuries unless the Slyde is used with proper cervical spinal precautions, for example backboard, C-collar, and appropriate strapping.
- **Slyde is for use in one (1) evacuation incident.**

Introduction Continued


CAUTION

Read the manual carefully before using the unit. Always follow correct operation procedures as outlined in this manual. Keep this manual on file at all times for reference.

- Do not store Slyde device or stairwell belay system in direct sunlight or in a moist environment.
- Keep Slyde and equipment out of direct contact with flames.
- Slyde must be stored where the temperature range is between 50 to 95 degrees Fahrenheit (10 to 35 degrees Celsius).

ENGLISH

SLYDE PRODUCT ILLUSTRATION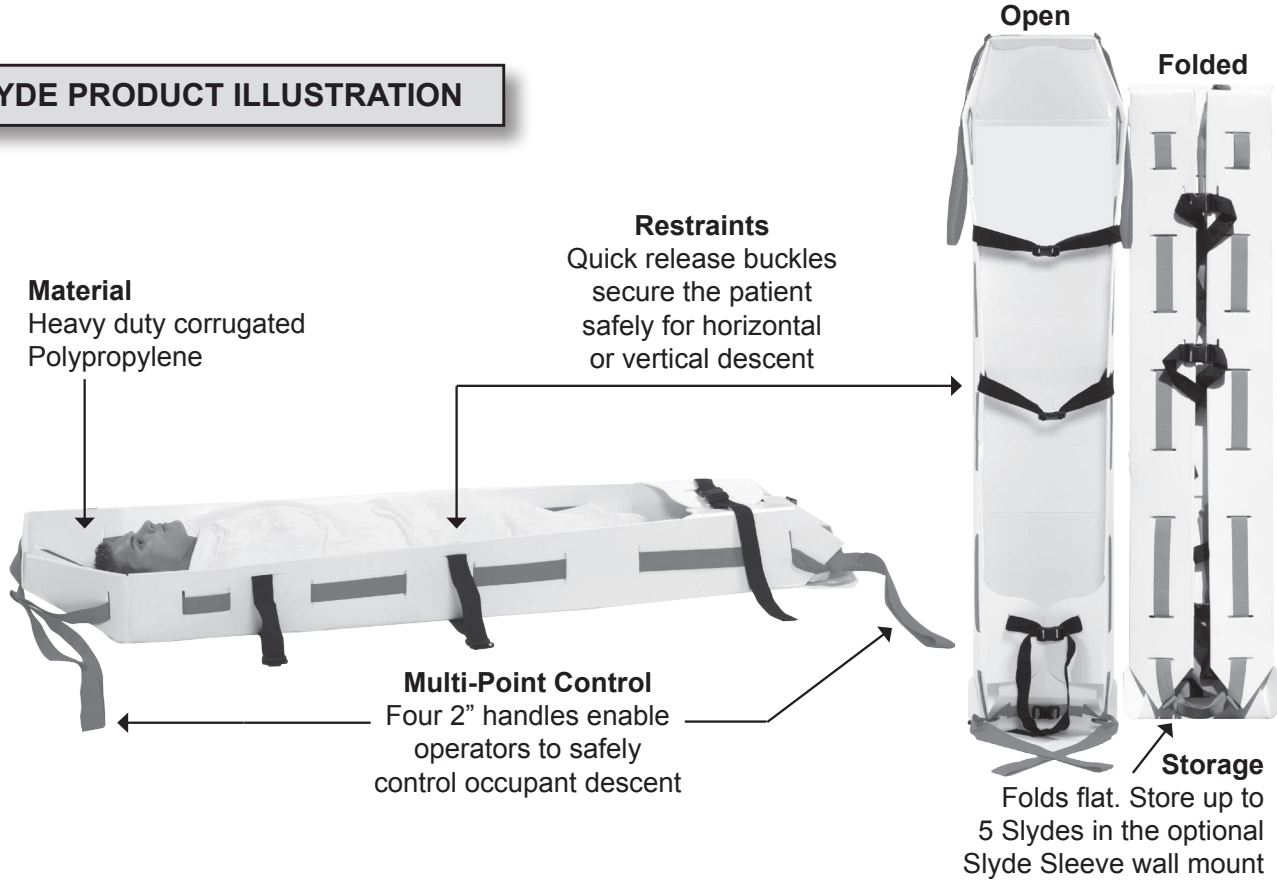

SPECIFICATIONS

belluscura™		6400-000-000 Evacuation Slyde	6410-000-000 Bariatric Evacuation Slyde	6400-020-000 Stairwell Belay System <small>*Sold Separately</small>
 Safe Working Load		500 lb (227 kg)	800 lb (363 kg)	5000 lb (2,267 kg)
Height		8.0" (20.3 cm)	8.0" (20.3 cm)	
Width		19.0" (48.3 cm)	24.0" (61.0 cm)	
Length		85.0" (215.9 cm)	85.0" (215.9 cm)	Rope - 30.0' (9.1 m)
Folded Height		2.0" (5.1 cm)	2.0" (5.1 cm)	
Weight		9.0 lb (4.1 kg)	11.0 lb (5.0 kg)	3.0 lb (1.4 kg)

Product Inspection

The condition of the Slyde is the responsibility of the owner. It is important that the Slyde is working properly before the product is put into service. Have a qualified person use the following list and the operating instructions to check Slyde before the product is put into service.

- _____ Unpack the cartons and check all items for proper operation.
- _____ Inspect Slyde and accessories thoroughly upon receipt for any damage that may have occurred during shipping and to ensure proper operation.
- _____ Inspect Slyde and accessories before each use for wear, fraying, or other damage.


ENGLISH

Assembly Instructions

These instructions are printed on the inside of each Evacuation Slyde and Bariatric Evacuation Slyde. This is an illustration of the Bariatric Evacuation Slyde.

Bariatric Evacuation Slyde

Assembly & Occupant Securing Instructions

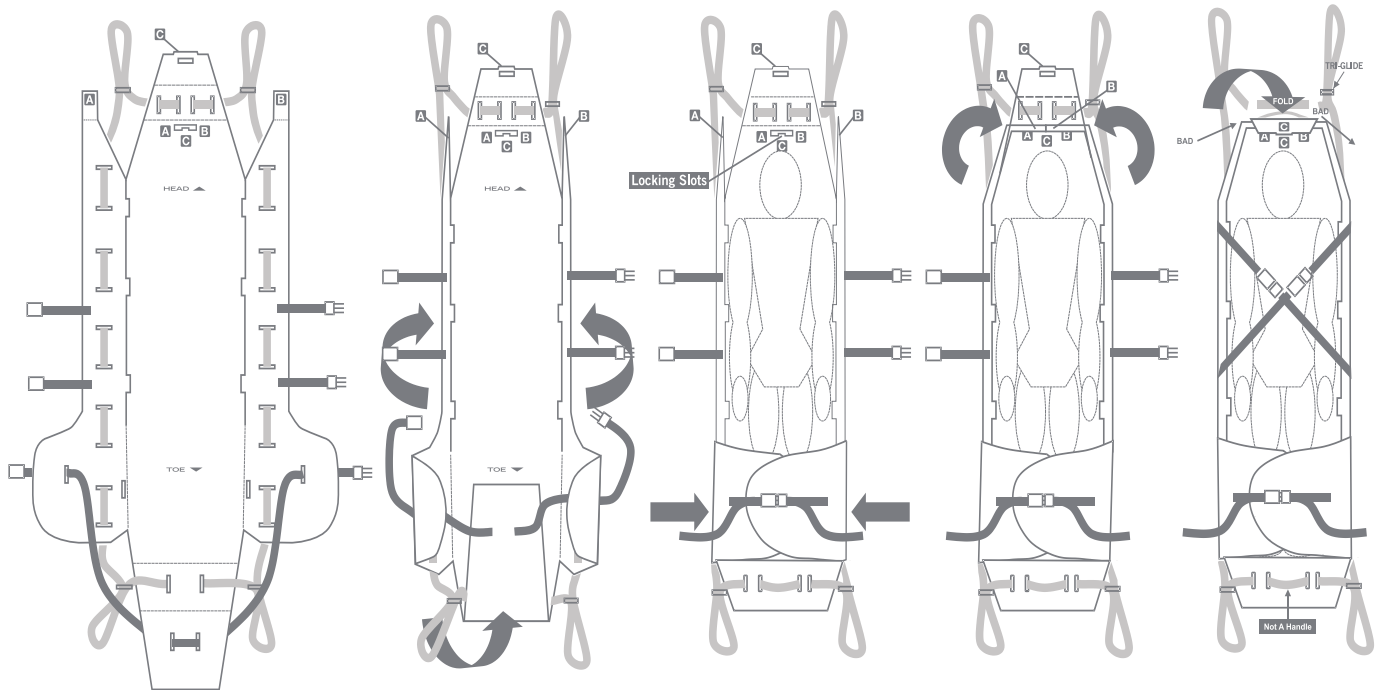1. Place Bariatric Evacuation Slyde flat on the floor or ground.

2. Fold sidewalls up to be vertical and fold the foot panel up over itself.

3. Capture the foot panel by folding over the extended sidewall panels near the feet. Secure the foot assembly by clipping the strap and adjusting it to be snug.

4. Form the head panel by folding the side wall extensions toward each other. The "A" and "B" tabs should align and index into the "A" and "B" slots.

5. Capture the folded and indexed "A" and "B" side wall extensions by folding the bottom "C" flap up and cover them. Lock the "C" tab by pushing it through the "C" locking slot. Secure the occupant and evacuate.

FOR TRAINED PROFESSIONAL USE ONLY

REF MODEL 6400-000-000 Evacuation Slyde

MAX. 500 LB (227 kg)

REF MODEL 6410-000-000 Bariatric Evacuation Slyde

MAX. 800 LB (363 kg)

INTENDED USE:

Slyde is an emergency sled for safely evacuating non-ambulatory residents or patients [occupant] from any multi-floor building using the stairway.

STORAGE:

Store Slyde in a dry, sunlight free environment 50° - 95°F

CAUTION:

Improper usage of the Evacuation Slyde or any accessory can cause injury to the occupant or the operator. Operate the Evacuation Slyde and accessories as described in its operation manual.

Covered by one or more of the following patents: US 7,422,220 B2; US 7,699,324 B2; EP 2,057,057 B8; CA2654308

Transporting An Occupant Using Slyde

These instructions outline the proper techniques for safely securing and transporting an occupant using the Slyde.

WARNING

- Do not operate Slyde, Stairwell Belay System, Stairwell Belay Box or other accessories without proper training.
- Do not exceed the 500 lb (227 kg) weight capacity of Evacuation Slyde or the 800 lb (363 kg) weight capacity of Bariatric Evacuation Slyde.
- Do not use Slyde if any bodily fluids are present. Inspect Slyde for fluids prior to use. Do not attempt to clean; dispose of equipment if fluids are present.
- Inspect Slyde and stairwell belay equipment at least annually and before each use for wear, fraying or other damage. If device or equipment is damaged, discontinue use.
- Slyde is not designed for use with suspected or known cervical, spinal, or fracture injuries unless the Slyde is used with proper cervical spinal precautions, for example backboard, C-collar, and appropriate strapping.
- To reduce the risk of injury to the occupant or operator, operators should never attempt to transport an occupant load greater than what they can safely lift.
- Water, ice, and debris on the stairs can affect operator footing and proper operation of Slyde. To reduce the risk of injury, clear path or consider an alternate route.
- Post evacuation, make sure that the patient is stable on a level surface.

NOTE (occupant in bed)

- If the occupant is currently in bed, place the bed in its lowest position, and lock the bed wheels, if applicable.
- Wrap the bed sheets around the occupant from both sides. Include pillows or blankets for the occupant's head and feet to provide additional occupant comfort.

TRANSFER THE OCCUPANT TO SLYDE

1. Unfold Slyde.
2. Log roll the occupant onto their side.
3. Place Slyde underneath the backside of the occupant.
4. Roll the occupant onto Slyde and place them in the middle of Slyde.
5. Construct the head end and foot end as shown. [See assembly instructions on page 4].
6. Fasten and tighten the safety straps.

WARNING

- The safety straps should be fastened diagonally, where possible, to prevent strangulation of shorter occupant [Caption 5 of the Assembly Instructions].
- Always use all restraint straps to secure the occupant to Slyde. An unrestrained occupant may fall from Slyde and be injured.
- The occupant must be secured on Slyde before transferring Slyde from the bed to the floor.

NOTE (occupant in bed)

- Pull the foot end of the mattress to the floor to create a ramp.
- While keeping your back in a neutral, ergonomic upright position, slide Slyde down the mattress to the floor.
- If the mattress is attached to the bed frame, use the device side handles to ease the occupant to the floor by lowering the foot end first followed by the head end.

PULL SLYDE WITH OCCUPANT TO THE NEAREST EMERGENCY EXIT

- Two operators may pull from the foot end by using the two handles as shown [Fig. 1].

Figure 1

Descending Stairs Using Evacuation Slyde

WARNING

- To reduce the risk of injury, transporting the occupant on stairs requires a minimum of two operators, one at the head and one at the foot of the Slyde. If more people are required to safely control Slyde, position operators around the edge of the Slyde using the handholds.
- At least two trained operators are required when using the Stairwell Belay System. More operators may be necessary depending on an occupant's size. See **page 7** as a reference for properly positioning operators.

ENGLISH

Two Person Descent:

- a. The foot end operator and the head end operator grasp both handles firmly.
- b. The foot end operator pulls Slyde three quarters of the way off of the first stair.
- c. The head end operator leans back to support the occupant's weight.
- d. Descend the stairs at a controlled speed. Stay toward the inside rail of the stairs to allow for an easier turn on the landing.
- e. Begin turn once Slyde is halfway on the landing. Reference the occupant's hip as a pivot point.
- f. Repeat the process on the next set of stairs.

Three Person Descent:

- a. Similar to the two-person descent, except that the two head end operators each take one handle.

Descending Stairs Using Bariatric Evacuation Slyde

WARNING

The Stairwell Belay System must be used when transporting bariatric patients 500 - 800 lb (227-363 kg)

To reduce the risk of injury when transporting bariatric occupants 500-800 lb (227-363 kg), strict observance of all precautions and safety procedures are necessary. Belluscura recommends the following equipment and manpower configurations for safe transport.

WARNING

- To reduce the risk of occupant or operator injury, the minimum number of operators listed in the table on **page 7** must be used.
- To reduce the risk of injury to the occupant or operator, operators should never attempt to transport occupant loads greater than what they can safely lift.

Additional Parts and Accessories List

Part name	Part Number
Evacuation Slyde Sleeve for 5 Slyde	6400-010-001
Bariatric Evacuation Slyde Sleeve for 5 Slyde	6410-010-001
Stairwell Belay System	6400-020-000
Stairwell Belay Box	6400-030-000

For parts: USA: +1 786-703-7996
Toll Free International Customer Service: +1 844-883-0940
Email: customerservice@belluscura.com


Operator Belay Configuration

NOTE: The numbers around the perimeter of the Bariatric Evacuation Slide indicate the number of operators and positions required during bariatric patient transport.


WARNING


500 LB - 600 LB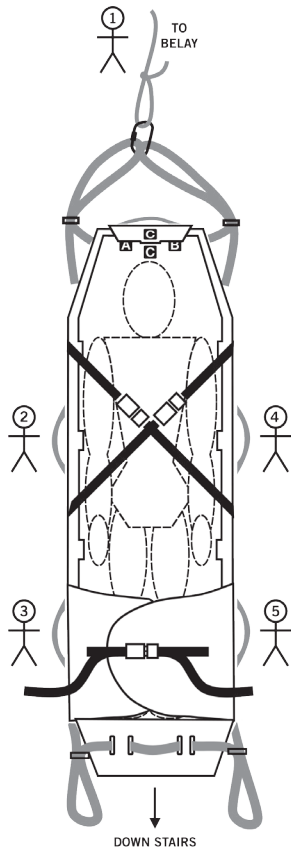

for patients weighing 500 - 800 LB

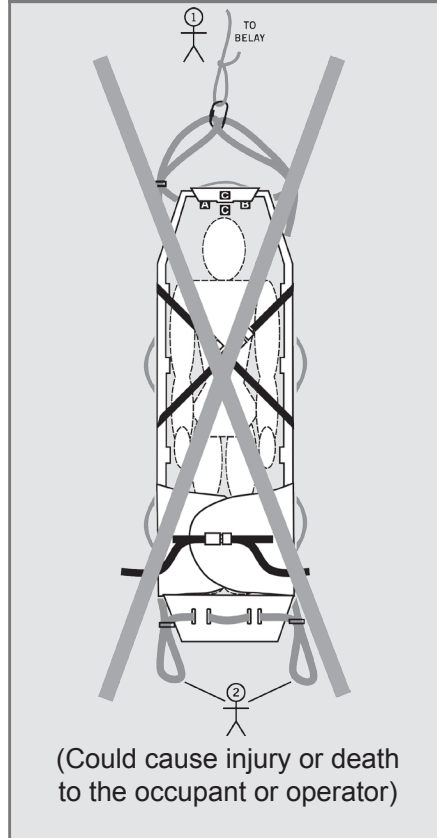600 LB - 800 LB

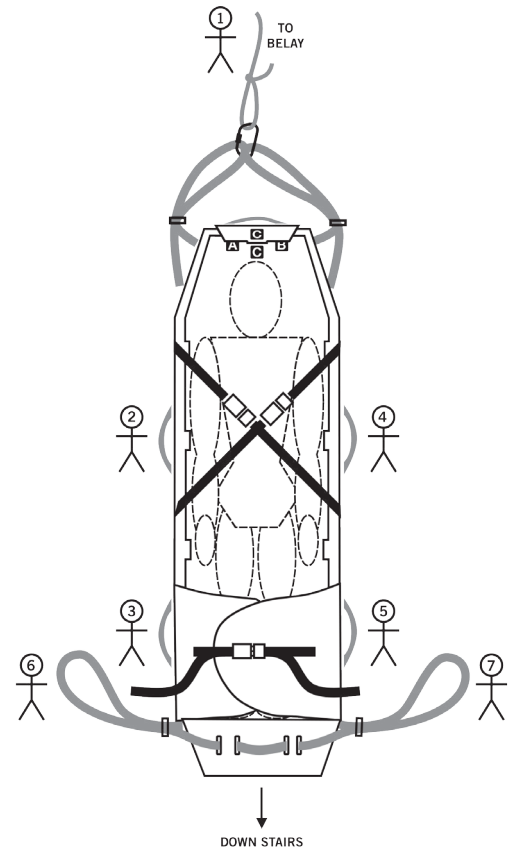ENGLISH

WARNING

See the table below for minimum number of operators for equipment and the occupant's combined weight.

Equipment & Occupant Combined Weight	Minimum Number of Operators
< 500 pounds	2
500 - 600 pounds	5
600 - 800 pounds	7

Warranty

Belluscura products are warranted to be free from defects in material and workmanship for a period of one year.

Belluscura repairs are warranted for 90 days from the date of repair. This limited warranty applies only when you use and care for the product as described in the instructions provided with the product. Improper use and care may void this warranty. The warranty period begins the date the product was first purchased by you as the original end-user or the date you as the original end-user receive it if you have proof of the delivery date. This Limited Warranty is not transferable or assignable in any manner. Shipping charges are not covered by this limited warranty. Belluscura is not liable for shipping damages or damages sustained through using the product. Non-Belluscura products sold by Belluscura retain the product manufacturer's original warranties and guarantees. Such warranties and guarantees, if any, are solely those of the product manufacturer. Belluscura offers no warranties or guarantees of any kind additional to those of the product manufacturer, nor does Belluscura assume any liability for products manufactured by others. In addition to our standard warranty, extended warranties are also available for many products. At the time you purchase your Belluscura product you can purchase an extended warranty for your certain products for an additional fee. These extended warranties include service and repair for the term of the extended warranty and can also include preventative maintenance services. **Call us at +1-786-703-7996 or email us at info@belluscura.com** for more information and a quote for your service.

LIMITATION OF LIABILITY

If a product is proven defective, Belluscura will repair or replace it, or, at its option, refund the item's purchase price. In no event is Belluscura liable for more than the selling price of the product. The purchaser accepts these terms in lieu of all damages. Except for the Limited Warranty above, Belluscura makes no other warranties, either expressed or implied. Belluscura makes no implied warranties of merchantability and fitness for a particular purpose for its own product or for the products of others. In no event will Belluscura be liable for any indirect or consequential damages sustained in connection with the delivery, use, or performance of its products. Under breach of limited warranty any legal action for breach of the limited warranty must be filed within one year of the date the breach is or should have been discovered.

LIMITED WARRANTY CLAIMS AND DISPUTES


Contact Belluscura Customer Relations promptly if you receive a product from Belluscura that you suspect is defective. A representative will assist you with the claim procedure. Before sending any product to Belluscura obtain a return authorization number from Belluscura Customer Relations.

15455 Dallas Parkway, Suite 600, Addison, TX 75001, USA

Oxford Centre for Innovation, New Road, Oxford, UK OX1 1BY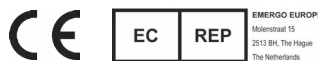

For parts or technical assistance: USA: +1 786-703-7996
Toll Free International Customer Service: +1 844-883-0940
Email: customerservice@belluscura.com

© Copyright 2016 Belluscura LLC All Rights Reserved.

Evacuation Slyde
REF Model 6400-000-000
Bariatric Evacuation Slyde
REF Model 6410-000-000

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© Copyright 2016 Belluscura LLC Todos los derechos reservados.
© Copyright 2016 Belluscura LLC Tous droits réservés
© Copyright 2016 Belluscura LLC Alle Rechte vorbehalten.
© Copyright 2016 Belluscura LLC Tutti i diritti riservati.
© Copyright 2016 Belluscura LLC Alle rechten voorbehouden.