# EVACUATION SLYDE®

ENGLISH

## SLYDE® STAIRWELL BELAY SYSTEM OPERATION MANUAL (MODEL #IC5030)

### SLYDE STAIRWELL BELAY SYSTEM

Offers complete control and safety for evacuation. Controls the descent of the Standard and Bariatric Evacuation Slyde.

#### ANCHOR CARABINER

Connects the stairwell belay device and stairwell belay rope to a firm anchor point.

#### ANCHOR POINT RUNNER/SLING

Wraps around a firm anchor point and connects a second time to the anchor carabiner.

#### STAIRWELL BELAY DEVICE

Provides friction to control the descent.

#### STAIRWELL BELAY ROPE

30' of rope to cover two flights of stairs; controls patient descent.

#### SLYDE CARABINER

Attaches to Evacuation Slyde handles.

*The Stairwell Belay System is for use with the Standard or Bariatric Evacuation Slyde.*

For parts or technical assistance contact: [info@dqeready.com](mailto:info@dqeready.com)

# OPERATION MANUAL

The Slyde Stairwell Belay System includes a rope, stairwell belay device, anchor point runner/sling, an anchor carabiner and Slyde carabiner.

## ⚠ WARNING

Only trained operators, trained in stairwell belay use, should use the Stairwell Belay System. Do not use the Stairwell Belay System for hoisting. To reduce the risk of injury, keep clothing, hair, fingers, & other body parts clear of the Belay System.

**Prior to use please read this Slyde Stairwell Belay System Operation Manual.**

*(The Stairwell Belay System is mandatory with the Bariatric Evacuation Slyde during stairwell descent)*

## TO USE THE STAIRWELL BELAY SYSTEM:

Figure 1

Figure 2a

Figure 2b

Figure 2c

Figure 2d

Figure 3a

1. Inspect equipment prior to use. See front page for the Stairwell Belay System configuration. Stairwell belay equipment comes pre-assembled (see **Figure 1**).
2. Determine a firm anchor point in the stairwell. As examples use accessible fixed handrail, vertical support beam or the anchor in the optional Stairwell Belay Box (DQE Model IC5031).

## ⚠ WARNING

Never use a person as an anchor point. Ensure that the anchor point is secure before using. Inadequate clearance may result in the inability to control the speed of the descent.

3. For a firm anchor point, such as a fixed handrail or support beam, wrap the free end of the colored anchor point runner around the anchor point and attach it back into the anchor carabiner and lock (see **Figure 2a**). When using the optional Stairwell Belay Box the anchor carabiner will be attached to the P38 anchor bolt in the box (see **Figure 2b**). Alternatively, you can run the free end of the anchor point runner through the P38 anchor bolt (see **Figure 2c**). Unlock and re-lock the anchor carabiner through the free end of the anchor point runner.

## ⚠ WARNING

To avoid the risk of patient or operator injury, make sure that both carabiners are locked by twisting the thumb lock mechanism before use.

4. Attach the rope to both handles at the head end of Evacuation Slyde/Bariatric Evacuation Slyde by using the Slyde carabiner (see **Figure 2d**).  
**Note:** If extra slack is present in the handle straps, take the left side handle strap at the head end of the sled and feed it down and through the front loop and back up to the top of the sled to attach it to the Slyde carabiner as shown in (see **Figures 3a - 3d**). Repeat the same process for the right side handle strap.

Figure 3b

Figure 3c

Figure 3d

Figure 4

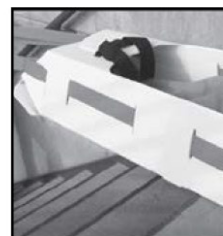
Figure 5

Figure 6

5. Now that the stairwell belay is attached to its anchor point and the Slyde carabiner is attached to the Evacuation Slyde/Bariatric Evacuation Slyde head end handles, the system is ready for use to control the descent of the patient down the stairs.  
**Note:** Communication between all operators, specifically between the stairwell belay operator and the foot end operator, is critical. Operators must communicate all actions to one another.
6. The stairwell belay operator wraps the rope around their back and takes out the slack in the rope (see **Figure 4**).
7. The foot end operator pulls Evacuation/Bariatric Evacuation Slyde three-quarters of the way off the first stair (see **Figure 5**).
8. The stairwell belay operator removes the slack in the rope, between them and the Slyde, to hold the weight of the patient (see **Figure 6**).
9. The stairwell belay operator slowly releases the rope through the stairwell belay device to control the speed of descent until the Evacuation Slyde/Bariatric Evacuation Slyde reaches the lower landing.
10. The stairwell belay operator removes the Stairwell Belay System and moves to the lower landing to help make the corner turn. Repeat steps 2-10 on the next set of stairs.

## ⚠ WARNING

To avoid the risk of patient and/or operator injury, keep a firm grip on the stairwell belay rope at all times to maintain control over the descent.

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