EVACUATION SLYDE®

ENGLISH

SLYDE® EVACUATION SLED OPERATION MANUAL

(SLYDE STANDARD MODEL #IC5010, SLYDE BARIATRIC MODEL #IC5020)

For parts or technical assistance
contact: info@dqeready.com

OPERATION MANUAL

TABLE OF CONTENTS

ENGLISH

Symbols and Definitions	2
Introduction	2
Intended Use	2
Specifications	3
Product Inspection	4
Assembly & Occupant Securing Instructions	4
Transporting An Occupant Using Slyde	5
Descending Stairs Using Evacuation Slyde	6
Descending Stairs Using Bariatric Evacuation Slyde	6
Belay Operator Configuration	6
Additional Parts and Accessories List	7

SYMBOLS AND DEFINITIONS

The words **WARNING**, **CAUTION** and **NOTE** carry special meanings and should be carefully reviewed.

Safe Working Load

Single Use

WARNING

Alerts the reader about a situation, which if not avoided, could result in death or serious injury. It may also describe potential serious adverse reactions and safety hazards.

CAUTION

Alerts the reader of a potentially hazardous situation, which if not avoided, may result in minor or moderate injury to the user or occupant or damage to the equipment or other property. This includes special care necessary for the safe and effective use of the device and the care necessary to avoid damage to a device that may occur as a result of use or misuse.

NOTE

This provides special information to make maintenance easier or important instructions clearer.

INTRODUCTION

This manual will assist you with the operation and maintenance of Evacuation Slyde (Standard and Bariatric). Thoroughly read this manual before using the equipment or beginning maintenance. Your staff should be trained on safe operation with written procedures.

INTENDED USE

The Slyde is an emergency sled for safely evacuating non-ambulatory residents or patients [occupants] weighing up to 500 lb (227 kg) using the Standard Evacuation Slyde or 800 lb (363 kg) using the Bariatric Evacuation Slyde from any multi-floor building using the stairway.

WARNING

- Do not modify the Slyde. Modifying the equipment can cause unpredictable operation resulting in injury to the occupant or operator. Modifying the Slyde will also void its warranty.
- Slyde is not designed for use with suspected or known cervical, spinal, or fracture injuries unless the Slyde is used with proper cervical spinal precautions, for example backboard, C-collar, and appropriate strapping.
- **Slyde is for use in one (1) evacuation incident.**


CAUTION

Read the manual carefully before using the unit. Always follow correct operation procedures as outlined in this manual. Keep this manual on file at all times for reference.

- Do not store Slyde device or stairwell belay system in direct sunlight or in a moist environment.
- Keep Slyde and equipment out of direct contact with flames.
- Slyde must be stored where the temperature range is between 50 to 95 degrees Fahrenheit (10 to 35 degrees Celsius).

INTRODUCTION CONTINUED

EVACUATION SLYDE PRODUCT ILLUSTRATION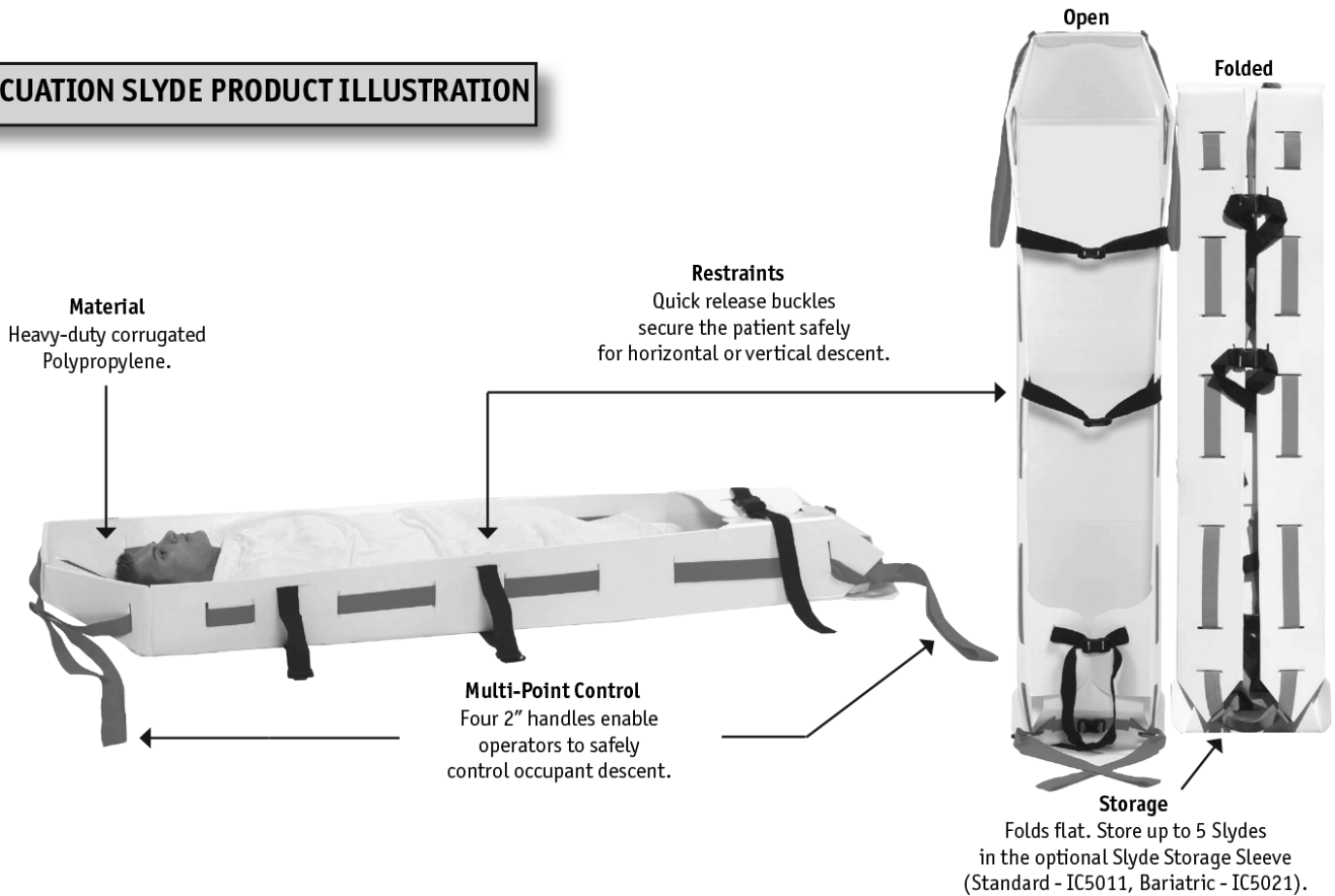

ENGLISH

SPECIFICATIONS

EVACUATION SLYDE		Evacuation Slide - Standard (IC5010)	Evacuation Slide - Bariatric (IC5020)	Slyde Stairwell Belay System (IC5030) *Sold Separately
	Safe Working Load	500 lb (227 kg)	800 lb (363 kg)	5,000 lb (2,267 kg)
	Height	8" (20.3 cm)	8" (20.3 cm)	
	Width	19" (48.3 cm)	24" (61 cm)	
	Length	85" (215.9 cm)	85" (215.9 cm)	Rope - 30' (9.1 m)
	Folded Height	2" (5.1 cm)	2" (5.1 cm)	
	Weight	9 lb (4.1 kg)	11 lb (5 kg)	3 lb (1.4 kg)

OPERATION MANUAL

PRODUCT INSPECTION


The condition of the Slyde is the responsibility of the owner. It is important that the Slyde is working properly before the product is put into service. Have a qualified person use the following list and the operating instructions to check the Slyde before the product is put into service.

ENGLISH

- _____ Unpack the cartons and check all items for proper operation.
- _____ Inspect Slyde and accessories thoroughly upon receipt for any damage that may have occurred during shipping and to ensure proper operation.
- _____ Inspect Slyde and accessories before each use for wear, fraying, or other damage.

ASSEMBLY & OCCUPANT SECURING INSTRUCTIONS

These instructions are printed on the inside of each Evacuation Slyde.

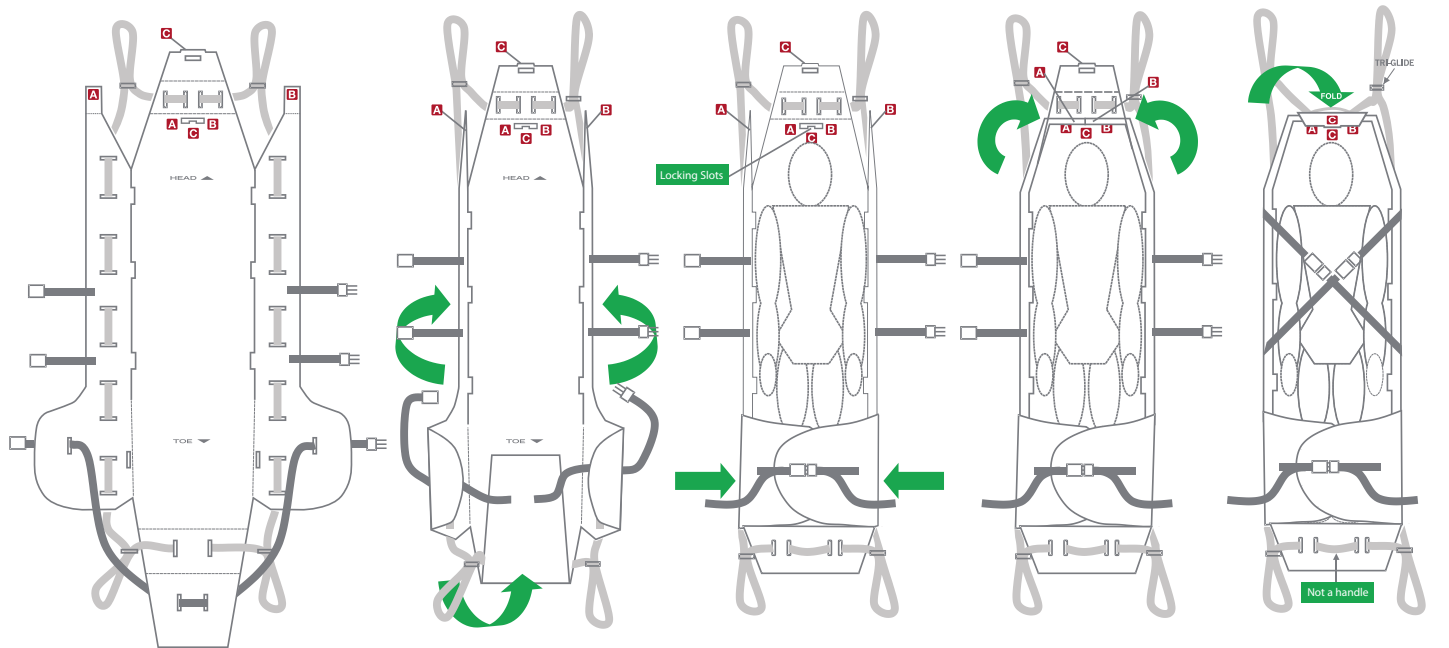1. Place Evacuation Slyde flat on the floor or ground. Unclip black straps and unfold Slyde as shown.

2. Fold sidewalls up to be vertical and fold the foot panel up over itself.

3. Capture the foot panel by folding over the extended sidewall panels near the feet. Secure the foot assembly by clipping the strap and adjusting it to be snug.

4. Form the head panel by folding the side wall extensions toward each other. The "A" and "B" tabs should align and index into the "A" and "B" slots.

5. Capture the folded and indexed "A" and "B" side wall extensions by folding the bottom "C" flap up and cover them. Lock the "C" tab by pushing it through the "C" locking slot. Secure the occupant and evacuate.

**FOR TRAINED
PROFESSIONAL
USE ONLY**

WEIGHT CAPACITY

EVACUATION SLYDE - STANDARD (IC5010)

500 lb (227 KG)

EVACUATION SLYDE - BARIATRIC (IC5020)

800 lb (363 KG)

INTENDED USE: Slyde is an emergency sled for safely evacuating non-ambulatory residents or patients [occupant] from any multi-floor building.

STORAGE: Store Slyde in a dry, sunlight free environment 50° - 95°F.

CAUTION: Improper usage of the Evacuation Slyde or any accessory can cause injury to the occupant or the operator. Operate the Evacuation Slyde and accessories as described in the operation manual.

Covered by one or more of the following patents: US 7,422,220 B2; US 7,699,324 B2; EP 2,057,057 B8; CA2654308

TRANSPORTING AN OCCUPANT USING SLYDE

These instructions outline the proper techniques for safely securing and transporting an occupant using the Slyde.

WARNING

- Do not operate Slyde, Stairwell Belay System, Stairwell Belay Box or other accessories without proper training.
- Do not exceed the 500 lb (227 kg) weight capacity of Evacuation Slyde or the 800 lb (363 kg) weight capacity of Bariatric Evacuation Slyde.
- Do not use Slyde if any bodily fluids are present. Inspect Slyde for fluids prior to use. Do not attempt to clean; dispose of equipment if fluids are present.
- Inspect Slyde and stairwell belay equipment at least annually and before each use for wear, fraying or other damage. If device or equipment is damaged, discontinue use.
- Slyde is not designed for use with suspected or known cervical, spinal, or fracture injuries unless the Slyde is used with proper cervical spinal precautions, for example backboard, C-collar, and appropriate strapping.
- To reduce the risk of injury to the occupant and/or operator, operators should never attempt to transport an occupant load greater than what they can safely lift.
- Water, ice, and debris on the stairs can affect operator footing and proper operation of Slyde. To reduce the risk of injury, clear path or consider an alternate route.
- Post evacuation, make sure that the patient is stable on a level surface.

NOTE (OCCUPANT IN BED)

- If the occupant is currently in bed, place the bed in its lowest position, and lock the bed wheels, if applicable.
- Wrap the bed sheets around the occupant from both sides. Include pillows or blankets for the occupant's head and feet to provide additional occupant comfort.

TRANSFER THE OCCUPANT TO SLYDE

1. Unfold Slyde. **[Illustration #1 on page 4]**
2. Log roll the occupant onto their side.
3. Place Slyde underneath the backside of the occupant.
4. Roll the occupant onto Slyde and place them in the middle of Slyde.
5. Fold sidewalls up. **[Illustration #2 on page 4]**
6. Construct the foot end and then head ends as shown. **[See assembly instructions on page 4]**
7. Fasten and tighten the safety straps. **[Illustration #5 on page 4]**

WARNING

- The safety straps should be fastened diagonally, where possible, to prevent strangulation of shorter occupant **[Illustration #5 of assembly instructions on page 4]**.
- Always use all restraint straps to secure the occupant to Slyde. An unrestrained occupant may fall from Slyde and be injured.
- The occupant must be secured on Slyde before transferring Slyde from the bed to the floor.

NOTE (OCCUPANT IN BED)

- Pull the foot end of the mattress to the floor to create a ramp.
- While the operator keeps their back in a neutral, ergonomic upright position, slide the Evacuation Slyde down the mattress to the floor.
- If the mattress is attached to the bed frame, use the Slyde side handles to ease the occupant to the floor by lowering the foot end first followed by the head end.

PULL SLYDE WITH OCCUPANT TO THE NEAREST EMERGENCY EXIT

- Two operators may pull from the foot end by using the two handles as shown **[Fig. 1]**.

Figure 1

OPERATION MANUAL

DESCENDING STAIRS USING THE STANDARD EVACUATION SLYDE

WARNING

- To reduce the risk of injury, transporting the occupant on stairs requires a minimum of two operators, one at the head and one at the foot of the Slyde. If more people are required to safely control the Slyde, position operators around the edge of the Slyde using the green strap handholds.
- At least two trained operators are required when using the Stairwell Belay System. More operators may be necessary depending on an occupant's size. See page 7 as a reference for properly positioning operators.

TWO PERSON DESCENT

1. The foot end operator and the head end operator grasp both handles firmly. One operator grasps both green strap handles at the head end. The other operator grasps both green strap handles at the foot end.
2. The foot end operator pulls Slyde three quarters of the way off of the first stair.
3. The head end operator leans back to support the occupant's weight.
4. Descend the stairs at a controlled speed. Stay toward the inside rail of the stairs to allow for an easier turn on the landing.
5. Begin turn once Slyde is halfway on the landing. Reference the occupant's hip as a pivot point.
6. Repeat the process on the next set of stairs.

THREE PERSON DESCENT

1. Similar to the two-person descent, except that the two head end operators each take one green strap handle.

DESCENDING STAIRS USING THE BARIATRIC EVACUATION SLYDE

WARNING

- The Stairwell Belay System (IC5030) (*sold separately*) must be used when transporting bariatric patients 500 - 800 lb (227-363 kg).

To reduce the risk of injury when transporting bariatric occupants 500-800 lb (227-363 kg), strict observance of all precautions and safety procedures is necessary. DQE recommends the following equipment and manpower configurations for safe transport.

WARNING

- To reduce the risk of occupant or operator injury, the minimum number of operators listed in the table below must be used.
- To reduce the risk of injury to the occupant and/or operator, operators should never attempt to transport occupant loads greater than what they can safely lift.

BELAY OPERATOR CONFIGURATION

WARNING

See the table to the right for minimum number of operators for equipment and the occupant combined weight.

Equipment & Occupant Combined Weight	Minimum Number of Operators
< 500 lb < 227 kg	2
500 - 600 lb 227 - 272 kg	5
600 - 800 lb 272 - 336 kg	7


BELAY OPERATOR CONFIGURATION CONTINUED

NOTE

The figures around the perimeter of the Bariatric Evacuation Sledge indicate the number of operators and positions required during bariatric patient transport.


500 LB - 600 LB

227 KG - 272 KG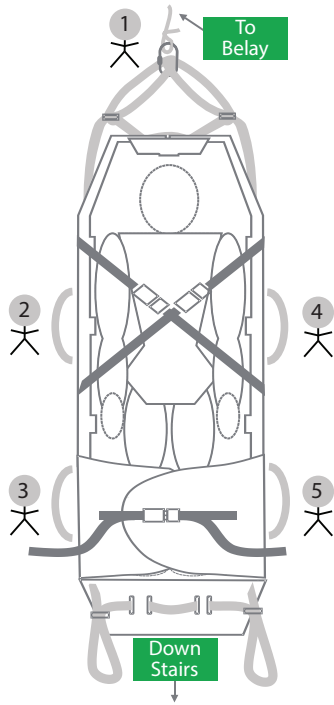


600 LB - 800 LB

272 KG - 363 KG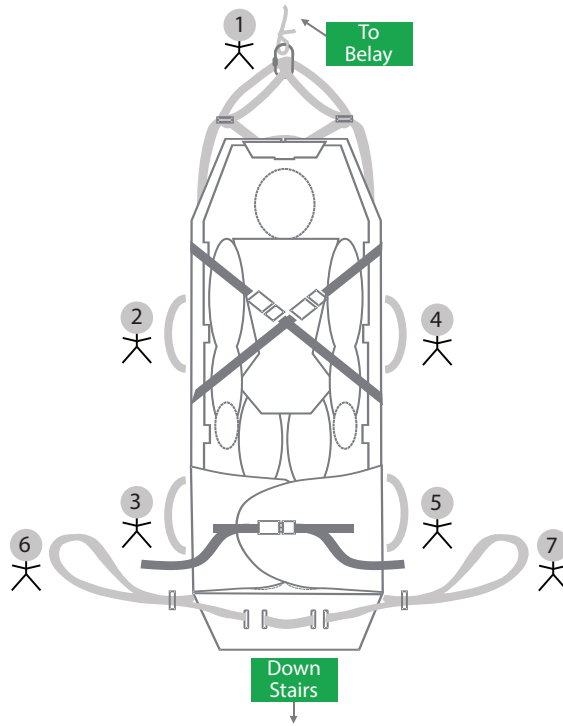

⚠ WARNING
INSUFFICIENT MANPOWER FOR OCCUPANTS WEIGHING 500 - 800 LB.

Could cause injury or death to the occupant or operator.

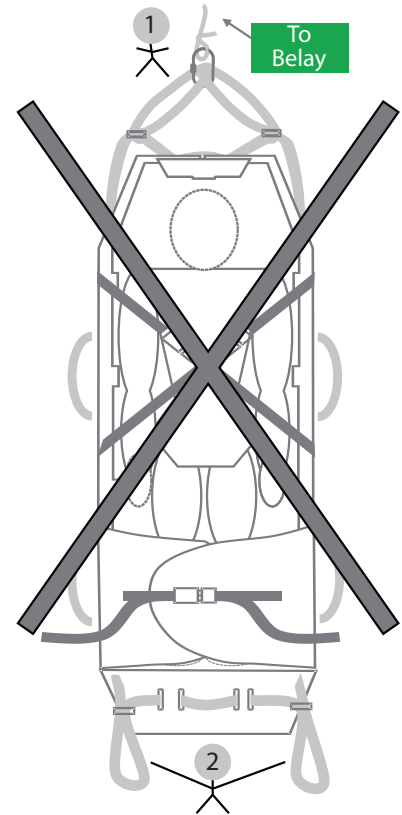ENGLISH

ADDITIONAL PARTS AND ACCESSORIES LIST

Part Name	Part Number
Slyde® Evacuation Sled Storage Sleeve - Standard	IC5011
Slyde® Evacuation Sled Storage Sleeve - Bariatric	IC5021
Slyde® Stairwell Belay System	IC5030
Slyde® Stairwell Belay Storage Box	IC5031

CONTACT DQE FOR PARTS OR TECHNICAL ASSISTANCE
 800-355-4628 | info@dqeready.com

EVACUATION SLYDE®

SLYDE® EVACUATION SLED MANUAL DE INSTRUCCIONES

(SLYDE STANDARD MODEL #IC5010, SLYDE BARIATRIC MODEL #IC5020)

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